



GIRAFFE GAZETTE

AMAZING THERAPY!

OCTOBER 9, 2009

PUTTING OUR BETTER FOOT FORWARD

A Question and Answer Session About the Juxta-Fit™ Ankle-Foot Wrap



Now that we have over a year's worth of experience with our Juxta-Fit™ product line, we are better able to field your questions and concerns regarding the popular CircAid Medical® Juxta-Fit™ Ankle-Foot Wrap.

Look for future Q & A's that will strive to crystallize your understanding and simplify your fitting sessions with CircAid Medical Products. Read on for enlightening information!

Q: The Juxta-Fit Ankle-Foot Wrap (AFW) Closed Heel pouches at the heel, was it made wrong?

A: No; by nature of the material and shape of the heel, application at this point is *extremely important*. Improper application may cause the foot-wrap to appear too short in length and may cause difficulty with band engagement. Click here to see the proper engagement and eliminate the pouch.

Q: The ankle-strap is longer on the separate Juxta-Fit AFW than it is on the attached (to legging) AFW. Is this right?

A: Yes. When the AFW is separate, gapping may occur between the foot piece and the legging; therefore the ankle-band is longer to ensure coverage in these areas.

Q: Why won't the Juxta-Fit custom AFW work on an unusual shape, an ankle-size larger than 38 – 40cm, or masses occurring in this area that hang or create creasing?

A: Whenever there are complications with shape, compromises must be made. With pictures, it makes it clear what our actual limitations will be. There are cases where we will send the custom, but will back it up with an EZ Single-Band AFW and/or a skin fold kit if you decide that the Juxta-Fit AFW does not provide optimal coverage or compression.

Q: The custom AFW isn't applying compression to the base of the toes, I've checked the measurements I provided, and nothing has changed. What happened?

A: A common effect from a congested dorsum is what is referred to as a "buffalo hump" or an acute change in circumference measurements. When your patient presents with this shape, make sure you are alerting us (via a description, drawing, or picture that accompanies your measurements) so that we can send a shelf strap to provide additional compression coverage.

(See 'Juxta-Fit AFW' Page 2)

1 - 8 0 0 - C I R C A I D (2 4 7 - 2 2 4 3)

CircAid®
MEDICAL - PRODUCTS

JUXTA-FIT™ ANKLE-FOOT WRAP (CONTINUED FROM PAGE 1)

Q: Why isn't the Juxta-Fit Interlocking AFW offered for custom size feet?

The purpose of the Interlocking band system (at the heel) is to allow for various foot lengths. When making a custom AFW there is no need for the interlock because the garment is made to the foot length provided.

NEW T-3 M™ DISCOUNT PRICING: THE FREEDOM TO CHOOSE

On orders for any of the T-3 M™ 6-Packs or **orders for 12 or more T-3 M leggings** the quantity discounted price will apply*. Send us your request and include the name of the business here for the updated dealer price list.

*Wholesale customers only.

	T-3 M	Juxta-Lite
Medicare Coverage	↙	
Bridge Product	↙	↙
Inelastic & Adjustable	↙	↙
Interlocking Straps	↙	
Juxtapositioned Straps		↙
Breathe-O-Prene™ Silver iner		↙
Built-in Pressure System™	↙	↙
Low cost	↙	↙
Simple Measuring	↙	↙
Easy Care	↙	↙