


Leg Measurements		
Circumference Measurements		
Position	Left	Right
3: 25cm from medial malleolus		
2: 15cm from medial malleolus		
1: 5cm from medial malleolus		

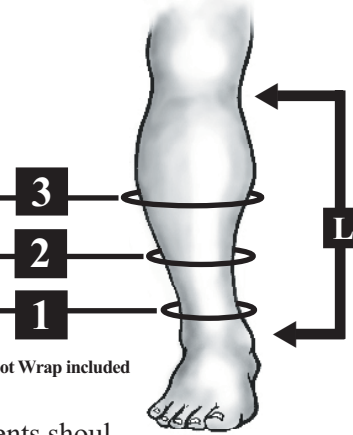

L	
Left	Right
medial length from prominent point of malleolus to knee crease.	

Patient:

- Should be standing.
- Limb should be at its smallest and limb size at time of measurements should be maintained to assure a proper fit of garment.

* EZ™ Single-Band Standard Ankle-Foot Wrap included

Select Product & Length
<input type="checkbox"/> 28cm Juxta-Fit™ Fits lengths L under 36cm
<input type="checkbox"/> 36cm Juxta-Fit™ Fits lengths L over 36cm
<input type="checkbox"/> 36cm Ready-Fit™ Length is adjustable
<input type="checkbox"/> 36cm Classic-Flex™ Length is adjustable

Circumference Measurement	S	M	L	XL	XXL†
1 5cm from ankle	15 - 23cm	20 - 28cm	25 - 33cm	30 - 38cm	33 - 41cm
2 15cm from ankle	20 - 28cm	25 - 33cm	30 - 38cm	36 - 46cm	43 - 53cm
3 25cm from ankle	30 - 38cm	36 - 43cm	41 - 48cm	46 - 56cm	53 - 64cm

Sizing Rules

Be sure measurements are accurate. When on borderline between sizes, consider amount of edema present & anticipated reduction. If significant edema is present and reduction is anticipated, select smaller size; if edema is minimal, select larger size. If measurements do **NOT** fall within **any [one] size category**, please see the custom measuring chart.

†For the Juxta-Fit™, if measurements fall within XXL size™ range, please consider ordering a custom Juxta-Fit™ for optimal fit.

Ankle-Foot Wrap (AFW) Selection
<input type="checkbox"/> Standard EZ (included) Single-Band 152x8cm
<input type="checkbox"/> Standard Juxta-Fit Closed heel
<input type="checkbox"/> Juxta-Lite Closed heel thin material
<input type="checkbox"/> Juxta-Fit Interlocking Open heel
<input type="checkbox"/> Custom EZ Single-Band 152x10cm (For larger feet out of size ranges)
<input type="checkbox"/> CircPlus II

Foot Measurements (Required, if other than Standard EZ™ Ankle-Foot Wrap is desired)																	
<table border="1"> <tr><th colspan="2">3</th></tr> <tr><th>Left</th><th>Right</th></tr> <tr><td></td><td></td></tr> <tr><td colspan="2">Ankle circumference around malleoli</td></tr> </table>	3		Left	Right			Ankle circumference around malleoli		<table border="1"> <tr><th colspan="2">1</th></tr> <tr><th>Left</th><th>Right</th></tr> <tr><td></td><td></td></tr> <tr><td colspan="2">Length from base of great toe to where foot meets ankle</td></tr> </table>	1		Left	Right			Length from base of great toe to where foot meets ankle	
3																	
Left	Right																
Ankle circumference around malleoli																	
1																	
Left	Right																
Length from base of great toe to where foot meets ankle																	
<table border="1"> <tr><th colspan="2">4</th></tr> <tr><th>Left</th><th>Right</th></tr> <tr><td></td><td></td></tr> <tr><td colspan="2">Foot length from base of toes to back of heel</td></tr> </table>	4		Left	Right			Foot length from base of toes to back of heel		<table border="1"> <tr><th colspan="2">2</th></tr> <tr><th>Left</th><th>Right</th></tr> <tr><td></td><td></td></tr> <tr><td colspan="2">Arch circumference</td></tr> </table>	2		Left	Right			Arch circumference	
4																	
Left	Right																
Foot length from base of toes to back of heel																	
2																	
Left	Right																
Arch circumference																	

Ankle-Foot Wrap Sizing			
Standard Juxta-Fit™ & Juxta-Lite™ AFW			
	S	M	L
1	> 9cm	> 9cm	> 9cm
2	19 - 24cm	24 - 29cm	29 - 34cm
3	< 36cm	< 36cm	< 41cm
4	> 16cm	> 16cm	> 16cm
Juxta-Fit™ Interlocking AFW			
	S	M	L
1	> 9cm	> 9cm	> 9cm
2	19 - 24cm	24 - 29cm	29 - 34cm
3	< 36cm	< 36cm	< 41cm
4	13 - 25cm	13 - 25cm	13 - 25cm
CircPlus™ II AFW			
	Std	L	XL
1	< 13cm	< 15cm	< 18cm
2	< 36cm	< 36cm	< 36cm
3	< 38cm	< 46cm	< 56cm