

# Juxta-Lite™, T-3 M™, T-3 EZ™, C-3 EZ™, T-3 Boot™, & C-3 Wrap™



## Standard Line Measuring and Sizing chart

### Sizing Rules

Be sure measurements are accurate. When on borderline between sizes, consider amount of edema present & anticipated reduction. If significant edema is present and reduction is anticipated, select smaller size; if edema is minimal, select larger size. If measurements do **NOT** fall within **any [one] size category**, a different product type (Juxta-Fit™, Ready-Fit™, Classic-Flex™) will need to be ordered. Size XL Short and XXL available in Juxta-Lite™ only.

Circumference Measurements	
Position	
3:	25cm from medial malleolus
2:	15cm from medial malleolus
1:	5cm from medial malleolus

4
medial length from prominent point of malleolus to the knee crease.
<b>Legging Length*</b>
Short : 28cm
Long : 33cm

5
Arch Circumference

### Patient:

- Should be standing.
- Limb should be at its smallest and limb size at time of measurements should be maintained to assure a proper fit of garment.
- Be sure to select a legging size and length for all product types. Select a foot size for T-3 M™ and Juxta-Lite™ only. Note: The XXL Juxta-Lite™ foot piece will come detached from the garment.

	Measurements		Small	Medium	Large	XL	Juxta-Lite™ Only			
	Left	Right					XXL			
<b>1</b> 5cm			19 - 22cm	22 - 27cm	25 - 32cm	30 - 37cm	35 - 42cm			
<b>2</b> 15cm			21 - 29cm	28 - 36cm	34 - 42cm	40 - 48cm	45 - 53cm			
<b>3</b> 25cm			25 - 36cm	33 - 44cm	41 - 51cm	48 - 58cm	53 - 64cm			
<b>4</b> *Length			Under 36cm Short	Over 36cm Long	Under 36cm Short	Over 36cm Long	Under 36cm Short	Over 36cm Long	Under 36cm Short	Over 36cm Long
<b>5</b> Arch Circumference (Foot Size)			19-26cm Small	24-31cm Med	19-26cm Small	24-31cm Med	19-26cm Small	24-31cm Med	19-26cm Small	24-31cm Med

### Foot Piece (Ankle-Foot Wrap)

The T-3 M™ and Juxta-Lite™ feature an attached closed heel Juxta-Lite™ ankle-foot piece (AFW). The XXL Juxta-Lite™ includes a detached Ankle-Foot Wrap. The foot piece can accommodate the circumference ranges listed above (#5), and a foot length of 16cm or greater. The T-3 EZ™ includes an attached EZ single-band ankle-foot wrap and the T-3 Boot™ includes an attached CircPlus™ II ankle-foot wrap. Both of these foot pieces are adjustable and fit most. The C-3 EZ™ and C-3 Wrap™ do not include a foot piece and one must be ordered separately. If foot measurements fall outside of these ranges and you fit into a standard size legging please contact us for additional assistance with your order.