


Whole Leg EZ™ Measuring & Sizing Chart

Instructions:

1. Patient: a) should be standing, b) limb should be at its smallest, c) limb size at time of measurements should be maintained to assure proper fit of garment.
2. Enter circumference measurements at specified points in the boxes below.
3. Always use undersock when testing a size.

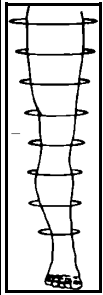


- H 80cm from center of ankle bone
- G 70cm from center of ankle bone
- F 60cm from center of ankle bone
- E 50cm from center of ankle bone
- D 40cm from center of ankle bone
- C 25cm from center of ankle bone
- B 15cm from center of ankle bone
- A 5cm from center of ankle bone

4. Be sure measurements are accurate.

When on borderline between sizes, consider amount of edema present and anticipated reduction.

If significant edema is present and reduction is anticipated, select smaller size; if edema is minimal select larger size. If measurements do **NOT** fall within **one** size range, call for guidance on selecting a size or use the custom measuring chart.

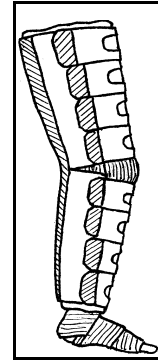

5. Length from medial malleolus to knee crease: _____

Note: Short and Long Whole Leg EZ™ will fit a below-knee length (ankle-to-knee) 32cm to 39cm.

6. Length medial malleolus to gluteal fold: _____

Two standard lengths available: 69cm (Short) and 76cm (Long)

Patient Measurement	Circumference Measurement	Small	Medium	Large	X-Large
	A 5cm from ankle	15-25 cm	20-30 cm	28-38 cm	38-53 cm
	B 15cm from ankle	20-35 cm	25-41 cm	33-48 cm	43-58 cm
	C 25cm from ankle	25-41 cm	30-46 cm	38-53 cm	48-63 cm
	D 40cm from ankle	25-46 cm	30-51 cm	38-58 cm	48-68 cm
	E 50cm from ankle	30-51 cm	35-56 cm	43-63 cm	53-74 cm
	F 60cm from ankle	43-63 cm	48-68 cm	56-76 cm	66-86 cm
	G 70cm from ankle	43-66 cm	48-71 cm	56-78 cm	66-89 cm
	H 80cm from ankle	46-68 cm	51-74 cm	58-81 cm	68-91 cm

7. Ankle-Foot Wrap (AFW) Sizing Chart

Model #	Size	Size Parameters
CircPlus I AFW (Black) 38210027	Standard fits up to	A. Arch: 35cm B. Foot Length: 12cm C. Ankle Circ: 38cm
CircPlus II AFW (Beige) 38230017	Large fits up to	A. Arch: 35cm B. Foot Length: 15cm C. Ankle Circ: 46cm
CircPlus II AFW (Beige) 38240017	XLarge fits up to	A. Arch: 40cm B. Foot Length: 18cm C. Ankle Circ: 56cm
CircPlus II AFW (Beige) 38290017	Custom (single band) fits over	A. Arch: 40cm B. Foot Length: 18cm C. Ankle Circ: 56cm

A. Arch Circ.

Left

Right

B. Foot Length

Left

Right

C. Ankle Circ.

Left

Right



Foot Length: Dorsum of foot from base of great toe to where foot meets ankle

Whole Leg EZ Legging System includes one pair undersocks, instructional DVD, and appropriate size AFW. CircAid Medical Products, Inc. assumes no liability for injury or damages resulting from any contraindicated use of these products.

Voice: (800)247-2243

Voice: (858)576-3550

Fax: (858)576-3555

CircAid®
MEDICAL PRODUCTS