

CIRCPLUS™

COMPONENTS OF COMFORT CAPRI™:

Nylon, Polyester, Lycra® and Cotton.

COMPONENTS OF COMPRESSION STOCKINGS:

Nylon and Spandex.

THESE PRODUCTS DO NOT CONTAIN LATEX.

INDICATIONS FOR USE: Pre and post vein procedure compression therapy,
Venous insufficiency

CONTRAINDICATIONS: Severe arterial insufficiency, acute infection.

This garment is not designed to be cut or modified in anyway. CircAid Medical Products, Inc. assumes no liability for injury or damages resulting from contraindicated use of our products. CircAid is a registered trademark and CircPlus and Comfort Capri are trademarks of CircAid Medical Products, Inc. All Rights Reserved. ©2011 CircAid Medical Products, Inc.

CIRCAID®

Making Compression Easier.

CP-DFU-102411-01



CircAid Medical Products, Inc.
9323 Chesapeake Drive, Suite B2
San Diego, CA 92123 U.S.A.
01-858-576-3550

DIRECTIONS FOR USE:

DONNING THE CIRCLPLUS™ COMPRESSION SYSTEM:

STEP 1. Roll the top edge of the stocking down until the heel of the stocking is visible. Grip the rolled edge of the stocking and slide the foot into the stocking.

STEP 2. Unroll the rest of the CircPlus™ Compression Stocking onto the leg.


STEP 3. Slide on the Comfort Capri™ and pull it up to the waist. The bottom elastic cuff of the Capri should overlap with the stocking.

REMOVING THE CIRCLPLUS™ COMPRESSION SYSTEM:

STEP 1. Slide the Comfort Capri™ off the legs.

STEP 2. Slide the CircPlus™ Compression Stockings off the leg.

Note: The CircPlus™ Compression System should never hurt. It should feel firm but comfortable. Both garments should be removed at night or while lying down.

 **If you are experiencing any pain from the garments, immediately remove them and consult your physician.**

Instructional videos are available at: www.circaid.com/circplus
If the compression stockings are hard to don, contact customer service at (800) CIRCAID (247-2243) for more information on alternatives to compression stockings.

CARE INSTRUCTIONS:



MACHINE WASH COLD



TUMBLE DRY LOW HEAT



DO NOT BLEACH



DO NOT DRY CLEAN



DO NOT WRING



DO NOT IRON

NOTE: To extend the life of your CircPlus™ System hand wash and drip or flat dry.

